


Life Whisperer

Application Reference Sheet

Common Tasks:

- How to add patients
- How to add a new cycle and upload embryos
- How to edit embryo information
- How to download reports

Task 1: How to add patients



Click the **Add Patient** button

1. Fill out the required information

2. Check the box

3. Click the **Save and Continue** button

Add Patient

IVF LW

Clinic (Required)

Patient ID (Required)

First Name

Last Name

Date Of Birth (Required)

Phone Number

Email Address

I confirm that we have received and securely stored this patient's executed Life Whisperer Patient Informed Consent and Life Whisperer Patient Privacy Notice.

Save and Continue **Save and Add New Patient** Cancel

Task 2a: Add a new cycle




Click the **New Cycle** button

Patient 001

Cycle Id	Assessment Types	Number of Images
001	Embryo Day 5 Genetics Embryo Day 5 Viability	8

Task 2b: Upload embryo images

1. Tick each box for **Assessment View**

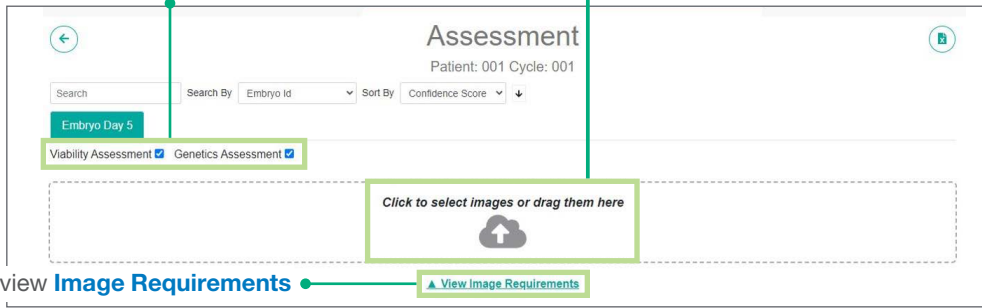


2. Click to view **Image Requirements**

3. To upload embryo images, you have 2 options:

- A Drag and drop images to this dotted box
- B Click the cloud icon and select the images from your computer



Task 3: Edit embryo information

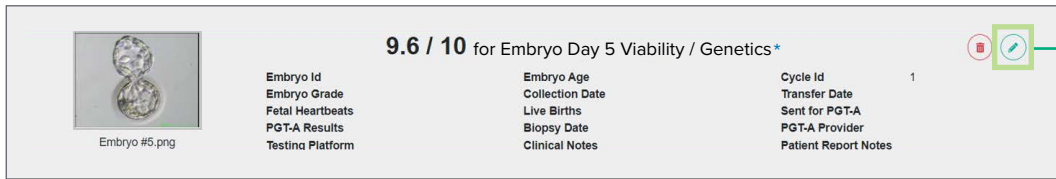

Click the **Edit** button


1. Embryo decision:

Click to **transfer**

Click to **freeze**

Click to **discard**


3. Click the **Save** button

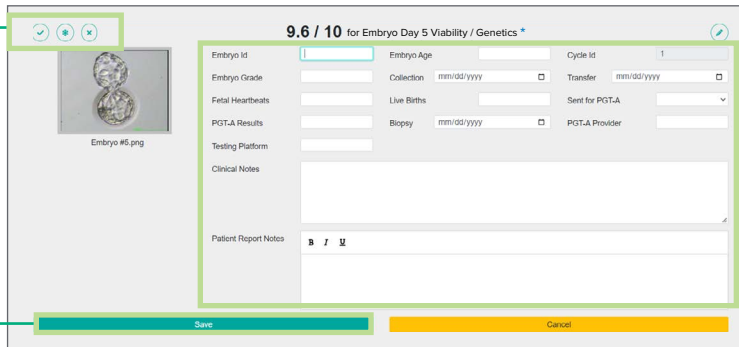

2. Fill in the embryo information as needed

* Each "Assessment" has it's own individual screen. Process is identical.

Task 4: Download Life Whisperer reports

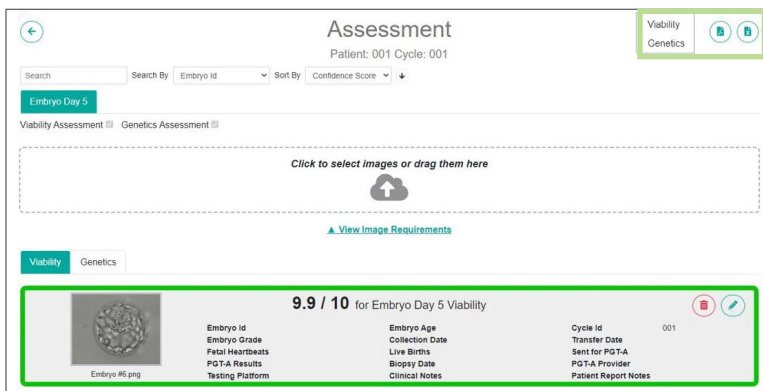

1. Patient PDF Report

Click the PDF icon to download Patient Reports. Reports are summaries of how Life Whisperer works and the final decisions and scores of the patient's embryos.

2. Clinic Excel Report

Click the Excel icon to download the excel sheet containing all of the embryo's information. This allows the clinic to review embryo information without logging into the Application.