## Modified Ham's F-10 Basal Medium HEPES

Catalog No. 99168

100 mL

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### ENGLISH

#### INTENDED USE

Modified Ham's F-10 Basal Medium HEPES is intended for use in assisted reproductive procedures which include retrieval, handling and transfer of gametes and embryos.

#### PRODUCT DESCRIPTION

Modified Ham's F-10 Basal Medium - HEPES consists of salts, amino acids, 20 mM HEPES, 25 mM bicarbonate buffers and other stable trace components. Supplementation with additional components (sodium lactate, sodium pyruvate, glutamine and gentamicin) are required prior to use. The recombination of the components is performed by the end user and results in a medium which has the same characteristics of pH and osmolality, as it would if made fresh.

#### COMPOSITION

The composition of the complete medium is a modification of the classical tissue culture Ham's F-10 formulation. This is a complex medium of inorganic salts, amino acids and vitamins.

Modified Ham's F-10 - HEPES (catalog #99168) has been specifically formulated without hypoxanthine, and with HEPES for buffering at ambient atmosphere (see section describing Buffer System).

#### QUALITY ASSURANCE

Modified Ham's F-10 Basal Medium - HEPES is a culture medium which is membrane filtered and aseptically processed according to manufacturing procedures which have been validated to meet a sterility assurance level (SAL) of  $10^{-3}$ .

Each lot of Modified Ham's F-10 Basal Medium HEPES is tested for:

- Endotoxin by Limulus Amebocyte Lysate (LAL) methodology
- Biocompatibility by Mouse Embryo Assay (one-cell)
- Sterility by the current USP Sterility Test <71>

All results are reported on a lot specific Certificate of Analysis which is available upon request.

#### BUFFER SYSTEM

Modified Ham's F-10 Basal Medium- HEPES uses a buffering system composed of a 20 mM HEPES (N-2-Hydroxyethylpiperazine-N-2-ethanesulfonic acid) and 25 mM bicarbonate combination. This buffering system provides optimum pH maintenance over the physiologic range (7.2 to 7.4) and does not require the use of a CO<sub>2</sub> incubator.

#### PROTEIN SUPPLEMENTATION

Modified Ham's F-10 Basal Medium-HEPES does not contain protein components. General laboratory practice includes protein supplementation when using this medium. The amount of protein supplementation may vary among laboratories and is dependent on the phase of processing/growing the gametes and embryos. Consult your individual laboratory protocols.

#### DIRECTIONS FOR USE

PREPARATION OF COMPLETE MODIFIED HAM'S F-10 BASAL MEDIUM

1. Allow bottle of Modified Ham's F-10 Basal Medium - HEPES (#99168) to equilibrate to room temperature.
2. Supplement basal medium aseptically with sterile solutions of sodium lactate, sodium pyruvate, glutamine and gentamicin to obtain final concentrations of 1 mM of each sodium lactate, sodium pyruvate and glutamine, as well as 10 µg/mL of gentamicin.
3. Recap the bottle and mix well by swirling the bottle containing the complete medium.

4. Optional components such as albumin or serum may be added at this point.

5. Note the date of supplementation on the bottle.

Note: Complete Modified Ham's F-10 - HEPES Medium should be tightly capped when used in a CO<sub>2</sub> incubator to avoid pH levels of 7.0 or less.

The complete liquid medium is then ready for use as a transport medium (pre-warmed at 37° C if desired), for the collection of human gametes and for embryo transfer to the patient. This medium is also used for procedures at ambient atmosphere (due to the HEPES buffering capabilities). These procedures include sperm washing and IntraCytoplasmic Sperm Injection (ICSI).

For additional details on the use of these products, each laboratory should consult its own laboratory procedures and protocols which have been specifically developed and optimized for your individual medical program.

#### STORAGE INSTRUCTIONS AND STABILITY

Store the unopened bottles of basal medium at 2° to 8° C. Do not freeze basal medium.

Modified Ham's F-10 Basal Medium - HEPES is stable until the expiration date shown on the bottle label when stored as directed.

The complete medium is stable until 57 days from date of supplementation as a "complete" medium.

Protect from fluorescent light.

#### PRECAUTIONS AND WARNINGS

Do not use any bottle of medium which shows evidence of particulate matter, cloudiness or is pink in color.

To avoid problems with contamination, handle using aseptic techniques, and discard any excess medium that remains in the bottle after the procedure is completed.

If basal medium is supplemented with antibiotic, appropriate precautions should be taken to assure patient is not sensitized to the antibiotic.

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