Colcemid[®] Solution

Catalog No. 9311

12 x 10 mL Kits

10 µg/mL in Hank's Balanced Salt Solution

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Irvine**Scientific**[®]

INTENDED USE

Colcemid® Solution is intended for *in vitro* diagnostic use.

PROPERTIES

Colcemid, N-deacetyl-N-methyl colchicine ($C_{21}H_{25}NO_5$) has a molecular weight of 371.42.¹ Colcemid is water soluble and sensitive to both light and heat. When dissolved in Hank's Balanced Salt Solution, Colcemid Solution has a pH of 7.3 ± 0.3 . Colcemid Solution arrests mitosis in both plant and animal cells,² preventing spindle formation during metaphase. However, Colcemid Solution is at least 30-40 times less toxic than colchicine.³

QUALITY ASSURANCE

A Material Safety Data Sheet and lot-specific Certificate of Analysis are available upon request.

SUGGESTED USES

1. Chromosome Analysis – amniotic cells after initial cultivation.⁴

Colcemid Solution arrests mitotic cultured cells in metaphase. Karyotyping for chromosome analysis is greatly facilitated when large numbers of metaphase cells are present in culture.

2. Cell Synchronization

Colcemid Solution aids in determining the mitotic activity index for cell synchronization studies.⁵ Through investigations, Colcemid Solution has been shown to inhibit HeLa and CHO cells at concentrations of 0.05 – 0.5 $\mu\text{g}/\text{mL}$.⁶

3. Polyploid Plants and Asexual Reproduction

Colcemid Solution produces an increase in the chromosome count. New plant species can be produced in this manner under suitable conditions.⁷ In asexual plant reproduction, the chromosome complement can be doubled under certain conditions.⁸

STORAGE INSTRUCTIONS AND STABILITY

Store tightly closed container at 2° – 8° C, protected from light.

When stored as directed, Colcemid is stable until the expiration date shown on the bottle label.

PRECAUTIONS AND WARNINGS

Colcemid Solution is mutagenic, tumorigenic, embryotoxic and teratogenic.

Colcemid® is a registered trademark of Ciba-Geigy Company, Basel, Switzerland.

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