Tyrode's Solution - Acidified

Catalog No. 99252

12 x 5 mL Kits

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Glossary of Symbols*:

	Catalog Number
	Lot Number
	Sterilized using aseptic processing techniques (filtration)
	Expiration: Year - Month - Day
	Caution, consult accompanying documents
	Consult instructions for use
	Storage Temperature: 15-30°C
	Do not re-use
	Do not re-sterilize
	Do not use if package is damaged
	Manufacturer
	U.S. Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.
	CE Mark
	Emergo Europe - Prinsessegracht 20 2514 AP The Hague The Netherlands

*Symbol Reference - EN ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labeling.

EU CAUTION: For Professional Use Only.

INDICATION FOR USE

Tyrode's Solution - Acidified is intended for use in assisted reproductive procedures which include gamete and embryo manipulation. These procedures include the use of Tyrode's Solution - Acidified for zona drilling.

DEVICE DESCRIPTION

Tyrode's Solution - Acidified facilitates assisted hatching by chemically drilling the zona pellucida. With a pH of 2.1-2.5, this solution's high acidity can dissolve the glycoproteins present in zona pellucida while avoiding damage to the embryo if applied carefully and in small quantities.

COMPOSITION:

<u>Salts and Ions</u>	<u>Energy Substrate</u>
Sodium Chloride	Glucose
Potassium Chloride	Water
Magnesium Chloride	WFI Quality
Sodium Phosphate	
Calcium Chloride	

QUALITY ASSURANCE

Tyrode's Solution - Acidified is membrane filtered and aseptically processed according to manufacturing procedures which have been validated to meet a sterility assurance level (SAL) of 10⁻³.

Each lot of Tyrode's Solution - Acidified is tested for: Endotoxin by Limulus Amebocyte Lysate (LAL) methodology
Sterility by the current USP Sterility Test <71>

All results are reported on a lot specific Certificate of Analysis which is available upon request.

DIRECTIONS FOR USE

Prepare embryo for assisted hatching by applying Tyrode's Solution - Acidified carefully to zona with an assisted hatching pipette until desired effect is achieved. Use only the amount of medium necessary to produce desired effect. Rinse embryo in neutral medium prior to embryo transfer.

For additional details on the use of these products, each laboratory should consult its own laboratory procedures and protocols which have been specifically developed and optimized for your individual medical program.

STORAGE INSTRUCTIONS AND STABILITY

Store the unopened bottles at room temperature: 15° to 30°C.

Do not freeze or expose to temperatures greater than 39°C.

PRECAUTIONS AND WARNINGS

This device is intended to be used by staff trained in assisted reproductive procedures. These procedures include the intended application for which this device is intended.

The user facility of this device is responsible for maintaining traceability of the product and must comply with national regulations regarding traceability, where applicable.

Do not use any bottle of medium which shows evidence of particulate matter, cloudiness or is not clear and colorless.

To avoid problems with contamination, handle using aseptic techniques, and discard any excess medium that remains in the bottle after the procedure is completed.

Information on known characteristics and technical factors that could pose a risk if the product were to be re-used have not been identified therefore the product is not to be used following the initial use of the container.

Tyrode's Solution - Acidified has a pH of 2.1 - 2.5. Appropriate precaution should be taken when handling this acidic product.

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