Water for Assisted Reproductive Technologies (A.R.T.) Use

Catalog No. 9307

1 L

For assisted reproductive procedures.

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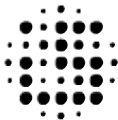
Irvine**Scientific**

INTENDED USE

Water for ART is intended for use in ART laboratory procedures requiring a non-pyrogenic high purity grade of water. Water for ART use is produced from sterile Ultra-Pure water that meets "Water for Injection" criteria in accordance with USP 24.NF 19.

PRODUCT DESCRIPTION

Sterile filtered distilled water. See Certificate of Analysis for specifications.

QUALITY ASSURANCE

Water for ART which is membrane filtered and aseptically processed according to manufacturing procedures which have been validated to meet a sterility assurance level (SAL) of 10^{-3} .

Each lot of Water for ART is tested for:

Endotoxin by LAL methodology

Biocompatibility by Mouse Embryo Assay

Sterility by the current USP Sterility Test <71>

All results are reported on a lot-specific Certificate of Analysis which is available upon request.

DIRECTIONS FOR USE

NOT INTENDED FOR INJECTION BY USUAL PARENTERAL ROUTES IN HUMANS OR ANIMALS.

For additional details on the use of this product, the laboratory should consult its own standard procedures and protocols which have been specifically developed and optimized for the individual medical program.

STORAGE INSTRUCTIONS AND STABILITY

Store the unopened bottles at room temperature for up to six months from date of manufacture. When stored as directed, the product is stable until the expiration date shown on the bottle.

1. Aspirate desired volume using aseptic procedures.
2. Once the desired volume has been removed from original bottle, do not return any volume of product to the original bottle.
3. Tightly reseal bottle.
4. After opening, bottle should be tightly re-sealed and stored at room temperature, away from light.

PRECAUTIONS AND WARNINGS

Do not use any bottle of water that shows evidence of particulate matter or microbial contamination.

To avoid problems with contamination, practice aseptic techniques and discard minimal amounts of excess product remaining in the bottle.

CAUTION: Federal (U.S.) law restricts this device to sale by or on the order of a physician.

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REFERENCES

Rinehart JS, Bavister BD and Gerrity, M: Quality Control in the in Vitro Fertilization Laboratory: Comparison of bioassay systems for water quality. *J Vitro Fert Embryo Transfer* 5:335-342, 1988.

Fleetham J and Mahadevan MM: Purification of water for in vitro fertilization and embryo transfer. *J Vitro Fert Embryo Transfer* 5:171-174. 1988.

Bavister BD, and Andrews JC: A rapid sperm motility bioassay procedure for Quality-Control testing of water and culture media. *J Vitro Fert Embryo Transfer* 5:67-75, 1988.

George MA, Braude PR, Johnson MH and Sweetnam DG: Quality control in the IVF laboratory: in-vitro and in-vivo development of mouse embryos is unaffected by the quality of water used in culture media. *Hum. Reprod.* 4:826-831, 1989.

Weimer KE, Anderson A and Stewart, B: The importance of water quality for media preparation. *Hum. Reprod. Supplement* 13, Supple. 4: 166-172, 1998.

Irvine**Scientific**

2511 Daimler Street, Santa Ana, California 92705-5588

Telephone: 1 949 261 7800 • 1 800 437 5706

Fax: 1 949 261 6522 • www.irvinesci.com

PN 40678 Rev. 2