

SYMS

Sealing system for CBS™ High Security straws

CE marked

DESCRIPTION / USE

Sealing system: Cryo Bio System has developed a thermal pulse sealing device for impermeable sealing of CBS™ High Security straws in ionic resin.

The sealing device is electronically operated by a push-button or foot pedal as desired.

INSTRUCTIONS FOR USE

- After the filling procedure with microaspirator, gently remove the transparent or yellow filling nozzle;
- Once this free part of the CBS™ straw is filled and sealed, there is no longer any risk of contamination;

- Insert an inner identification rod, or outer colored identification jacket;
- Seal the other end of the straw;
- Freeze according to a method appropriate for the type of product.

TECHNICAL SPECIFICATIONS

L x W x H: 230 x 220 x 170 mm

Weight: 6 kg

Power supply: 110 or 220 V / 60 or 50 Hz

