# FUJIFILM

# IrvineScientific

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### REFERENCES

Conaghan, J. et. al.: Effects of pyruvate and glucose on the development of human preimplantation embryos in vitro. *Journal of Reproduction & Fertility* (1993), 87-95.

Quinn, P. et. al.: Successful human in vitro fertilization using a modified human tubal fluid medium lacking glucose and phosphate ions. *Fertility and Sterility*, 1995; 63: 922-4.

Carrillo, A. et al.: Improved clinical outcomes for in vitro fertilization with delay of embryo transfer from 48 to 72 after oocyte retrieval: Use of glucose- and phosphate-free media. *Fertil Steril* 1998; 69: 329-34.

Pool, T.B. et. al.: Oocyte and Embryo Culture: Basic Concepts and Advances. In *Infertility and Reproductive Medicine*, Clinics of North America 9(2): 181-203 Assisted Reproduction Laboratory Considerations (J. May, ed.) 1998.

### FUJIFILM Irvine Scientific, Inc.

2511 Daimler Street, Santa Ana, California 92705 USA

Telephone: 1 949 261 7800 • 1 800 437 5706 Fax: 1 949 261 6522 • www.irvinesci.com

© 2019 FUJIFILM Irvine Scientific, Inc. All rights reserved. The FUJIFILM Irvine Scientific logo and P-1 are trademarks of FUJIFILM Irvine Scientific, Inc. in various jurisdictions. PN 40616 Rev.11

### ENGLISH

**EU Caution:** For Professional Use Only

### INDICATION FOR USE

P-1 Medium is intended for use in assisted reproductive procedures which include human gamete and embryo manipulation. These procedures include the use of P-1 as a culture medium through day 3 of development.

### DEVICE DESCRIPTION

Device Description- P-1 Medium is bicarbonate-based and is designed for procedures utilizing a CO<sub>2</sub> incubator. P-1 contains the antibiotic Gentamicin Sulfate (10 µg/mL).

### COMPOSITION

<u>Salts and Ions</u>	<u>pH Indicator</u>
Sodium Chloride	Phenol Red
Potassium Chloride	
Magnesium Sulfate	<u>Buffer</u>
Calcium Chloride	Sodium Bicarbonate
<u>Antioxidant</u>	<u>Antibiotic</u>
Sodium Citrate	Gentamicin Sulfate
<u>Amino Acids</u>	<u>Water</u>
Taurine	WFI Quality
<u>Energy Substrate</u>	
Sodium Pyruvate	
Sodium Lactate	

### QUALITY ASSURANCE

P-1 is a culture medium which is membrane filtered and aseptically processed according to manufacturing procedures which have been validated to meet a sterility assurance level (SAL) of 10<sup>-3</sup>.

Each lot of P-1 Medium is tested for:

- Endotoxin by Limulus Amebocyte Lysate (LAL) methodology
- Biocompatibility by Mouse Embryo Assay (one-cell)
- Sterility by the current USP Sterility Test <71>

All results are reported on a lot specific Certificate of Analysis which is available upon request.

### BUFFER SYSTEM

P-1 Medium uses sodium bicarbonate as a buffering system. This is specifically designed for use in a CO<sub>2</sub> incubator.

### PROTEIN SUPPLEMENTATION

P-1 Medium does not contain protein components. General laboratory practice includes protein supplementation when using this medium. The amount of protein supplementation may vary among laboratories and is dependent on the phase of processing/growing the gametes and embryos. Consult your individual laboratory protocols.

### DIRECTIONS FOR USE

P-1 Medium should be warmed to 37°C and equilibrated to the desired pH in a CO<sub>2</sub> incubator prior to use. When equilibrating in a CO<sub>2</sub> incubator, the bottle of P-1 Medium should be loosely capped to allow for the exchange of gas and pH equilibration.

After the medium has been pre-warmed and equilibrated in a CO<sub>2</sub> incubator, allow fertilization to occur in a culture dish that contains P-1 Medium supplemented with protein. Once fertilization occurs, the reproductive specialist should transfer the zygote/embryo into a new dish with pre-warmed equilibrated P-1 containing the desired concentration of protein for the growth phase. Allow embryo to grow until desired developmental stage (up to 3 days).

For additional details on the use of these products, each laboratory should consult its own laboratory procedures and protocols which have been specifically developed and optimized for your individual medical program.

### STORAGE INSTRUCTIONS AND STABILITY

Store the unopened bottles refrigerated at 2° to 8°C.

Do not freeze or expose to temperatures greater than 39°C.

Duration Following Bottle Opening:

Product should be used within (8) weeks from opening when stored under the recommended conditions of 2° to 8°C.

### PRECAUTIONS AND WARNINGS

This device is intended to be used by staff trained in assisted reproductive procedures. These procedures include the intended application for which this device is intended.

The user facility of this device is responsible for maintaining traceability of the product and must comply with national regulations regarding traceability, where applicable.

Do not use any bottle of medium which shows evidence of particulate matter, cloudiness or is not pale orange in color.

To avoid problems with contamination, handle using aseptic techniques, and discard any excess medium that remains in the bottle after the procedure is completed.

### CONTRAINDICATION

Product contains Gentamicin Sulfate. Appropriate precautions should be taken to ensure that the patient is not sensitized to the antibiotic.

### Glossary of Symbols\*:

Catalog Number

Lot Number

Sterilized using aseptic processing techniques (filtration)

Expiration:  
Year - Month - Day

Caution, consult accompanying documents

Consult instructions for use

Storage Temperature  
2-8°C

Do not resterilize

Do not use if package is damaged

Manufacturer

U.S Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

CE Mark

Emergo Europe - Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

\*Symbol Reference - EN ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labelling.

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