FUJIFILM

Irvine Scientific

Oil for Embryo Culture

Catalog No. 9305

100 mL, 500 mL

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REFERENCES

Johnson C, Hofmann G, and Scott R.: The use of oil overlay for in-vitro fertilization and culture. Assisted Reproduction Review 4:198-201, 1994.

Miller KF, Goldberg JM, and Collins RL.: Covering embryo cultures with mineral oil alters embryo growth by acting as a sink for an embryotoxic substance. J. Assist. Reprod. Genet., 11:342, 1995.

Provo MD, and Herr C.: Washed paraffin oil becomes toxic to mouse embryos upon exposure to sunlight. Theriogenology 49:214, 1998.

Glossary of Symbols*:

	Catalog Number
	Lot Number
	Sterilized using aseptic processing techniques (filtration)
	Expiration: Year - Month - Day
	Caution, consult accompanying documents
	Consult instructions for use
	Storage Temperature: 15-30°C
	Manufacturer
	Do not resterilize
	Do not use if package is damaged
	U.S. Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.
	CE Mark
	Emergo Europe - Prinsessegracht 20 2514 AP The Hague The Netherlands

*Symbol Reference - EN ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labeling.

ENGLISH

EU Caution: For Professional Use Only

INDICATION FOR USE:

Oil for Embryo Culture is intended for use as an overlay to cover small volumes of culture media during human embryo and gamete manipulation.

Not for use as an injectable product.

DEVICE DESCRIPTION

Sterile light mineral oil.

QUALITY ASSURANCE

Light mineral oil which is membrane filtered and aseptically processed according to manufacturing procedures which have been validated to meet a sterility assurance level (SAL) of 10⁻³.

Each lot of Oil for Embryo Culture is tested for:

- Endotoxin by LAL methodology
- Biocompatibility by Mouse Embryo Assay
- Human Sperm Survival Assay
- Sterility by the current USP Sterility Test <71>

All results are reported on a lot-specific Certificate of Analysis which is available upon request.

DIRECTIONS FOR USE

The use of an oil overlay is recommended when culturing in reduced volumes of media to prevent evaporation and to insulate the media from changes to osmolality and pH.

Oil should be warmed to 37°C and equilibrated in CO₂ incubator for several hours (or overnight) before use.

Washing of Oil for Embryo Culture is not required.

For additional details on the use of these products, each laboratory should consult its own laboratory procedures and protocols which have been specifically developed and optimized for your individual medical program.

STORAGE INSTRUCTIONS AND STABILITY

Store the unopened bottles away from light at room temperature.

Do not freeze or expose to temperatures above 39°C.

When stored as directed, product is stable until the expiration date shown on the bottle.

Duration Following Bottle Opening:

The product once opened has a eight week shelf life when stored under the recommended conditions of 15° to 30°C.

PRECAUTIONS AND WARNINGS

This device is intended to be used by staff trained in assisted reproductive procedures. These procedures include the intended application for which this device is intended.

The user facility of this device is responsible for maintaining traceability of the product and must comply with national regulations regarding traceability, where applicable.

Do not use any bottle of oil that shows evidence of particulate matter or microbial contamination.

Do not use any bottle in which the sterile packaging has been compromised.

To avoid problems with contamination, practice aseptic techniques and discard minimal amounts of excess product remaining in the bottle.

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© 2019 FUJIFILM Irvine Scientific, Inc. All rights reserved. The FUJIFILM Irvine Scientific logo is a trademark of FUJIFILM Irvine Scientific, Inc. in various jurisdictions. PN 40673 Rev.10