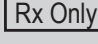
Refrigeration Medium - Test Yolk Buffer (TYB) with Gentamicin

Catalog No. 90129

20 x 5 mL Kits

For refrigeration of semen.
Zur Kühlung von Samen.
Per la refrigerazione del liquido seminale.
Para criopreservación de semen.
Pour la réfrigération du sperme.
Para a refrigeração de sêmen.
Για ψύξη σπέρματος.
Ke zchlazeni spermatu.
Til nedkøling af sæd.
Siittiöiden pakastukseen.
Spermas atdzesēšanai.
Voor het koelen van sperma.
Do schładzania nasienia.
Pentru refrigerarea spermei.
För kylskåpsförvaring av sperma.
Seemnevedeliku sügavkülmutamiseks.
Az onó hűtésére.
Spermai atvésintli.
Meniyi buzdolabında soğutmak için.
Na chladienie semena.
За охлаждане на семенна течност.
Za rashladivanje sjemena.
Ghar-refrigerazzjoni tas-semen.
Za hlajenje semenske tekočine.

Glossary of Symbols*:

	REF	Catalog Number
	LOT	Lot Number
	STERILE A	Sterilized using aseptic processing techniques (filtration)
		Expiration: Year - Month - Day
		Caution, consult accompanying documents
		Consult instructions for use
		Storage Temperature below -10°C
		Do not re-use
		Do not re-sterilize
		Do not use if package is damaged
		Manufacturer
	Rx Only	U.S. Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.
	CE 0050	CE Mark
	EC REP	Emergo Europe - Prinsessegracht 20 2514 AP The Hague The Netherlands

*Symbol Reference - EN ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labeling.

REFERENCES

- Jaskey DG, Cohen MR: *Twenty-four to ninety-six hour storage of human sperm in TEST-yolk buffer*. Fertil Steril 35:205, 1981.
- Bolanos JR, Overstreet JW, Katz DF: *Human sperm penetration of zona-free hamster eggs after storage of semen for 48 hours at 2°C to 5°C*. Fertil Steril 39:536, 1983.
- Johnson AR, Syms AJ, Lipschultz LI, Smith RG: *Conditions influencing human sperm capacitation and penetration of zona-free hamster ova*. Fertil Steril 41:603, 1984.
- Syms AJ, Johnson AR, Lipschultz LI, Smith RG: *Effect of aging and cold temperature storage of hamster ova as assessed in the sperm penetration assay*. Fertil Steril 43:766, 1985.
- Hirsch I, Gibbons WE, Lipschultz LI, Rossavik KK, Young RL, Poindexter AN, Dodson MG, Findley WE: *In vitro fertilization in couples with male factor infertility*. Fertil Steril 45:659, 1986.

FUJIFILM Irvine Scientific, Inc.

2511 Daimler Street, Santa Ana, California 92705 USA
Telephone: 1 949 261 7800 • 1 800 437 5706 Fax: 1 949 261 6522 • www.irvinesci.com

© 2019 FUJIFILM Irvine Scientific, Inc. All rights reserved. The FUJIFILM Irvine Scientific logo is a trademarks of FUJIFILM Irvine Scientific, Inc. in various jurisdictions PN 40681 Rev.7

ENGLISH

EU Caution: For Professional Use Only.

INDICATION FOR USE

Refrigeration Medium is intended for refrigeration of human semen for use in assisted reproductive procedures.

DEVICE DESCRIPTION

Component	Specification
TES.....	176 mM
Tris.....	80 mM
Dextrose.....	9 mM
Gentamicin Sulfate.....	10 µg/mL
Heat-inactivated egg yolk obtained from specific pathogen free (SPF) laying flocks.....	20 % (v/v)

COMPOSITION

<u>Copolymer Surfactant</u>	
TES Buffer	<u>Antibiotics</u>
Buffers	Gentamicin Sulfate
Trizma base	<u>Cryoprotectant</u>
<u>Protein</u>	Glycerol
Chicken Egg Yolk	<u>Water</u>
<u>Energy Substrates</u>	WFI Quality
Fructose	

QUALITY ASSURANCE

Refrigeration Medium is membrane filtered and aseptically processed according to manufacturing procedures which have been validated to meet a sterility assurance level (SAL) of 10⁻³.

Each lot of Refrigeration Medium is tested for:

- Endotoxin by Limulus Amebocyte Lysate (LAL) methodology
- Sperm Motility Recovery Assay
- Sterility by the current USP Sterility Test <71>

All results are reported on a lot specific Certificate of Analysis which is available upon request.

DIRECTIONS FOR USE

- Semen is collected following a two to three day abstinence period.
- Sample is allowed to liquefy at 37°C for 30 minutes.
- One vial of medium is thawed and brought to 37°C.
- The liquefied sample is transferred to a sterile, 15 mL, conical centrifuge tube, the volume determined and medium added dropwise until a 1:1 sample:medium ratio is achieved.

Note: Samples displaying high viscosity may require the additional step of repeated pipetting or passage through an 18 gauge needle to ensure thorough mixing.

- The sample-medium mixture is placed in a beaker or other suitable container of 37°C water.
 - The container is refrigerated at 2°C to 5°C to allow a slow cooling of the mixture (0.5°C/minute) for a maximum of 96 hours.
 - See references 2, 3 and 5 for exact methodological details.
- Note: Steps 1-4 can be performed at room temperature.

For additional details on the use of these products, each laboratory should consult its own laboratory procedures and protocols which have been specifically developed and optimized for your individual medical program.

STORAGE INSTRUCTIONS AND STABILITY

Store the unopened bottles at -10° C or colder.

Do not expose medium to repeated freeze-thaw cycles. If smaller aliquots are desired, thaw the product, aliquot into sterile labeled containers and freeze until time of use.

Refrigeration Medium is stable until the expiration date shown on the bottle or vial label when stored as directed.

Discard after use.

PRECAUTIONS AND WARNINGS

This device is intended to be used by staff trained in assisted reproductive procedures. These procedures include the intended application for which this device is intended.

The user facility of this device is responsible for maintaining traceability of the product and must comply with national regulations regarding traceability, where applicable.

Refrigeration Medium will appear opaque. Do not use any bottle of medium which shows evidence of particulate matter or is not yellow in color.

To avoid problems with contamination, handle using aseptic techniques and discard any excess medium that remains in the bottle or vial after the procedure is completed.

Do not use any bottle in which the sterile packaging has been compromised.

Information on known characteristics and technical factors that could pose a risk if the product were to be re-used have not been identified therefore the product is not to be used following the initial use of the container.

CONTRAINDICATION

Product contains Gentamicin Sulfate and egg yolk protein. Appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic or egg yolk protein.

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