FUJIFILM

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Za postopke asistirane reprodukcije.

REFERENCES

Biggers, JD. and Racowsky, C. *The development of fertilized human ova to the blastocyst stage in KSOM^{AA} medium: is a two-step protocol necessary?* RBMOnline, 5:133-140, 2002.

Pool, TB. *Recent advances in the production of viable human embryos in vitro.* RBMOnline, 4:294-302, 2002.

Biggers, JD. *Thoughts on embryo culture conditions.* RBMOnline, 4 (suppl.1):30-38, 2001.

Lane, M., Hooper, K., and Gardner, DK. *Effect of essential amino acids on mouse embryo viability and ammonium production.* J. Asst. Reprod. Genet. 18: 519-525, 2001

Devreker, F., Hardy, K., Van den Bergh, M., Vanin, AS., Emiliani, S., and Englert, Y. *Amino acids promote human blastocyst development in vitro.* Hum. Reprod. 16:153-163, 2001.

Devreker, F., Van den Bergh, M., Biramane, J., Winston, RML., Englert, Y., and Hardy, K. *Effects of taurine on human embryo development in vitro.* Hum. Reprod. 14: 2350-2356, 1999.

FUJIFILM Irvine Scientific, Inc.

2511 Daimler Street, Santa Ana, California 92705 USA

Telephone: 1 949 261 7800 • 1 800 437 5706 Fax: 1 949 261 6522 • www.irvinesci.com

© 2019 FUJIFILM Irvine Scientific, Inc. All rights reserved. The FUJIFILM Irvine Scientific logo and MultiBlast Medium are trademarks of FUJIFILM Irvine Scientific, Inc. in various jurisdictions
PN 40763 Rev.11

ENGLISH

EU Caution: For Professional Use Only

INDICATION FOR USE

MultiBlast Medium is intended for use in the culture of human embryos to the blastocyst stage of development. MultiBlast Medium is intended for use in culturing human embryos from Day 3 to blastocyst (up to Day 6).

DEVICE DESCRIPTION

MultiBlast Medium is formulated to provide a complex array of components required for ongoing development of embryos to the blastocyst stage. MultiBlast Medium is bicarbonate based and is designed for use in a CO₂ incubator. MultiBlast Medium contains the antibiotic Gentamicin Sulfate (10 µg/mL)

COMPOSITION

Salts and Ions

Sodium Chloride
Potassium Chloride
Magnesium Sulfate
Potassium Phosphate
Calcium Chloride

Energy Substrates

Glucose
Sodium Pyruvate
Sodium Lactate

Dipeptide

Alanyl-Glutamine

Amino Acids

Alanine
Arginine
Asparagine
Aspartic Acid
Glutamic Acid
Glycine
Histidine
Isoleucine
Leucine
Lysine
Methionine
Phenylalanine
Proline
Serine
Taurine
Threonine
Tryptophan
Tyrosine
Valine
Cystine

Antibiotic

Gentamicin Sulfate

pH Indicator

Phenol Red

Buffer

Sodium Bicarbonate

Antioxidant

Sodium Citrate

Water

WFI Quality

QUALITY ASSURANCE

MultiBlast Medium is membrane filtered and aseptically processed according to manufacturing procedures which have been validated to meet a sterility assurance level (SAL) of 10⁻³.

Each lot of MultiBlast Medium is tested for:

Endotoxin by Limulus Amebocyte Lysate (LAL) methodology
Biocompatibility by Mouse Embryo Assay (one-cell)
Sterility by the current USP Sterility Test <71>

All results are reported on a lot specific Certificate of Analysis which is available upon request.

BUFFER SYSTEM

MultiBlast Medium uses sodium bicarbonate as a buffering system. This is specifically designed for use in a CO₂ incubator.

DIRECTIONS FOR USE

PROTEIN SUPPLEMENTATION

MultiBlast Medium does not contain protein components. General laboratory practice includes protein supplementation when using this medium. The amount of protein supplementation can vary among laboratories and is dependent on the phase of processing/growing the gametes and embryos. Consult your individual laboratory protocols.

EQUILIBRATION

MultiBlast Medium (supplemented with protein) should be warmed to 37°C and equilibrated to the desired pH in a CO₂ incubator prior to use.

After achieving desired developmental stage at day 3, transfer embryo to a culture dish containing pre-warmed, equilibrated MultiBlast Medium which contains desired concentration of protein supplement. The reproductive

laboratory specialist may continue growth until transfer on day 4 to 6.

For additional details on the use of these products, the laboratory should consult its own standard procedures and protocols which have been specifically developed and optimized for the individual medical facility.

STORAGE INSTRUCTIONS AND STABILITY

Store the unopened bottles refrigerated at 2° to 8°C.

Do not freeze or expose to temperatures greater than 39°C.

Duration Following Bottle Opening:

The product once opened has an eight week shelf life when stored under the recommended conditions of 2° to 8°C.

PRECAUTIONS AND WARNINGS

This device is intended to be used by staff trained in assisted reproductive procedures. These procedures include the intended application for which this device is intended.

The user facility of this device is responsible for maintaining traceability of the product and must comply with national regulations regarding traceability, where applicable.

Do not use any bottle of medium which shows evidence of particulate matter, cloudiness or is not pale orange in color.

Do not use any bottle in which the sterile packaging has been compromised.

To avoid problems with contamination, handle using aseptic techniques and discard any excess medium that remains in the bottle after the procedure is completed.

Not for injection use.

CONTRAINDICATION

Product contains Gentamicin Sulfate. Appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic.

Glossary of Symbols*:

REF Catalog Number

LOT Lot Number

STERILE A Sterilized using aseptic processing techniques (filtration)

Expiration: Year - Month - Day

Caution, consult accompanying documents

Consult instructions for use

Storage Temperature 2-8°C

Do not resterilize

Do not use if package is damaged

Manufacturer

Rx Only

U.S. Caution: Federal law restricts this device to sale by or on the order of a licensed healthcare practitioner.

CE Mark 0050

Emergo Europe - Prinsessegracht 20 2514 AP The Hague The Netherlands

*Symbol Reference - EN ISO 15223-1, Medical devices – Symbols to be used with medical device labels, labeling.

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