

Life Whisperer Viability

Application Reference Sheet

Common Tasks:

- How to add patients
- How to add a new cycle and upload embryos
- How to edit embryo information
- How to download reports

Task 1: How to add patients



Click the **Add Patient** button

1. Fill out the required information



Patient

Clinic (Required) Test IVF

Patient ID (Required)

First Name

Last Name

Date of Birth (Required) mm/dd/yyyy

I confirm that we have received and securely stored this patients' executed Life Whisperer Informed Consent Sheet.

Save and Continue Save and Add New Patient Cancel

2. Check the box

3. Click the **Save and Continue** button


Task 2: Add a new cycle



Click the **New Cycle** button

Task 3: Upload embryo images

1. Check the box **Viability Assessment**

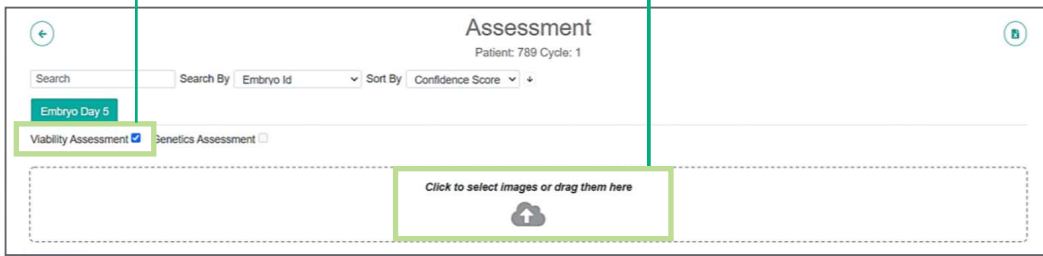


2. To upload embryo images, you have 2 options:

- A Drag and drop images to this dotted box
- B Click the cloud icon and select the images from your computer




Task 4: Edit embryo information

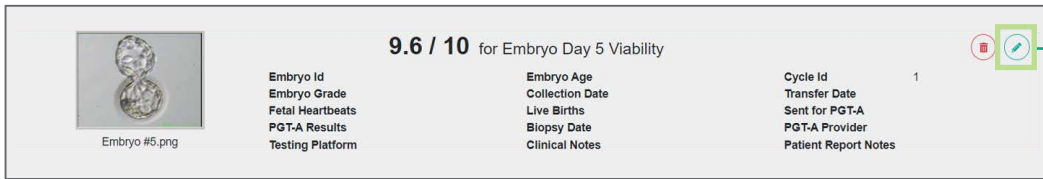

- Click the **Edit** button


1. Embryo decision:
 - Click to **transfer**
 - Click to **freeze**
 - Click to **discard**

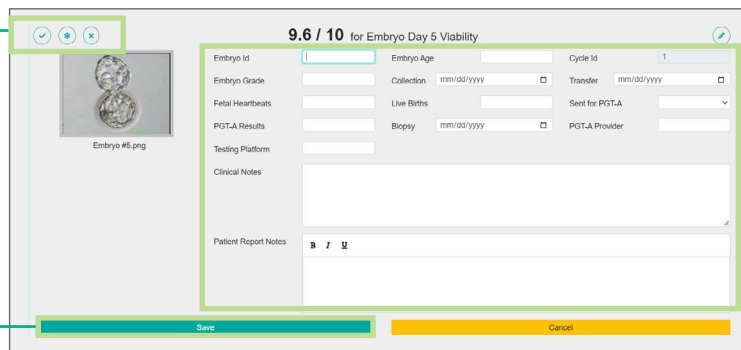

2. Fill in the embryo information as needed

3. Click the **Save** button

Task 4: Download Life Whisperer Viability reports

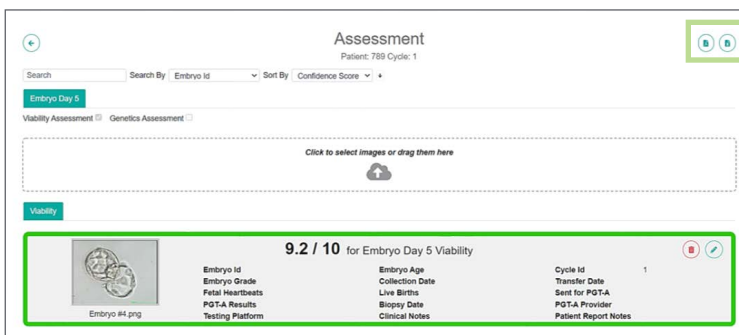

1. Patient PDF Report

Click the PDF icon to download the Patient Report. The Patient Report is a summary of how the Application works and the final decisions and scores of the patient's embryos.

2. Clinic Excel Report

Click the Excel icon to download the excel sheet containing all of the embryo's information. This allows clinic to review embryo information without logging into the Application.